



# WHEN USING THE AeroChamber MAX® Valved Holding Chamber (VHC)

## It's important to remember the following tips to gain the child's cooperation.



**N**ever force the mask on the child or attempt to treat while the child is crying.



**G**ive praise and rewards whenever the child is using the chamber correctly.



**P**ractice 'short' pretend treatments by having the mask near the child's face.



**U**se teddy bears, dolls or even adults to 'pretend' to use the chamber and make it look like a game.



**L**et the child play with the chamber while offering encouragement.



**G**ive the treatment when the child is happy, playing or sleeping.



**G**ive the child stickers to put on their chamber to make it their own.



Remember to read the instructions that accompany your new AeroChamber MAX® VHC!

™ and © are trademarks and registered trademarks of Monaghan Medical Corporation or an affiliate of Monaghan Medical Corporation



**monaghan™**  
Monaghan Medical Corp.  
5 Latour Ave., Suite 1600  
Plattsburgh, NY 12901  
www.monaghanmed.com  
1.800.833.9653  
PN 76109-03 12/08